

For In Vitro Diagnostic Use



Collection precautions:

Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Donor should NOT eat, drink, smoke or chew gum for 30 minutes before collecting oral sample.

Intended use: This product is designed for the collection and stabilization of DNA from human oral samples.

Contents: Contains 1 collection kit.

Warning and precautions: Choking hazard. Caution should be used when inserting sponge into the mouth.

Wash with water if liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com

Storage: 15-25°C

Summary and explanation of the kit:

ORACollect-DNA is a self-collection kit that provides the materials and instructions for collecting human oral samples.

Label legend:

- Catalog number
- In vitro diagnostic medical device
- CE Marking
- Manufacturer
- Authorized Representative
- Storage instructions
- Collect sample by (Use by)
- Lot number

ENGLISH

Instructions for sample collection:

	<p>Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.</p>
	<p>Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.</p>
	<p>Hold the tube upright to prevent the liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.</p>
	<p>Turn the cap upside down, insert the sponge into the tube and close cap tightly.</p>
	<p>Invert the capped tube and shake vigorously 15 times.</p>



For In Vitro Diagnostic Use



*Superior samples
Proven performance*

DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com



Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands
Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

ORACollect*-DNA is not available for sale in the United States.
*ORACollect is a registered trademark of DNA Genotek Inc.
Some DNA Genotek products may not be available in all geographic regions, contact your sales representative for details.
All DNA Genotek protocols, white papers and application notes, are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)
© 2019 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.
PD-PR-00188 Issue 5/2019-10