


USER INSTRUCTIONS

Read all instructions prior to collection

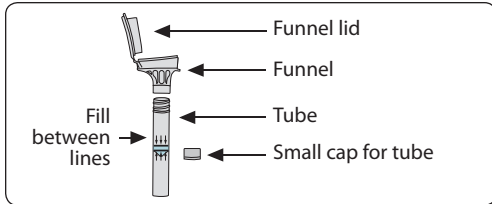

Collection precautions:

Do NOT eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.

Do NOT remove the plastic film from the funnel lid.

Do NOT spit above the top line on the tube (picture 1).

Intended use: For the collection of human DNA from saliva samples.

Contents: Kit contains stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com.

Small cap may pose a choking hazard.

Storage: 15°C / 30°C

Summary and explanation of the kit:


Oragene-ONE is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Label legend:


	Consult package insert
	Collect saliva by (Use by)
	In vitro diagnostic medical device
	Catalog number
	CE Marking
	Caution, consult instructions for use
	Storage instructions
	Authorized Representative
	Manufacturer
	Lot number

Procedure:


Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 5.

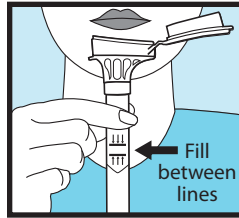


- 1 Spit a SMALL amount of saliva into the funnel. Gently tap the base of the tube on a flat surface to help the saliva move into the tube. Check that liquid saliva (not bubbles) level is between the lines as shown in picture #1.

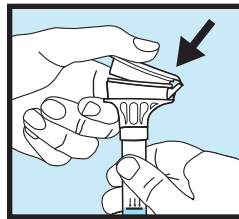


- 2 Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The tinted liquid in the lid will be released into the tube to mix with the saliva. Make sure that the lid is closed tightly.

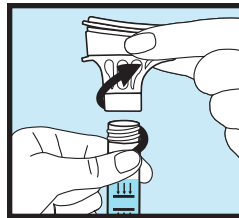

- 3 Hold the tube upright. Unscrew the funnel from the tube.

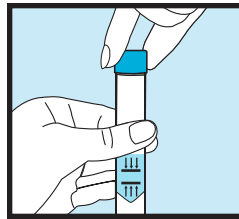

- 4 Use the small cap to close the tube tightly.

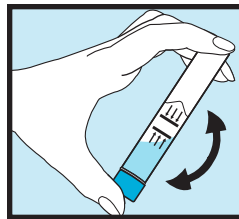

- 5 Shake the capped tube for 5 seconds. Discard or recycle the funnel.



For In Vitro Diagnostic Use

DNAgenotek



Made in Canada
 DNA Genotek Inc.
 3000 - 500 Palladium Drive
 Ottawa, ON, Canada K2V 1C2

Superior samples
Proven performance

Toll-free (North America): 1.866.813.6354
 Tel: +1.613.723.5757 • Fax: +1.613.723.5057
 info@dnagenotek.com
www.dnagenotek.com



Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

Oragene®-ONE is not available for sale in the United States.

*Oragene is a registered trademark of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions, contact your sales representative for details. All DNA Genotek protocols, white papers and application notes, are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)

© 2017 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.
 PD-PR-00260 Issue 5/2017-05