

## USER INSTRUCTIONS

Read all instructions prior to collection

### Collection precautions:

**Do NOT eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.**

**Do NOT remove the plastic film from the cap.**

**Intended use:** For the collection of human DNA from saliva samples.

**Contents:** Kit contains stabilizing liquid.

### Warnings and precautions:











Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at [www.dnagenotek.com](http://www.dnagenotek.com).

**Storage:** 15°C / 25°C

### Summary and explanation of the kit:


Oragene-DNA is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.
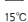
### Label legend:


	Consult package insert
	Collect saliva by (Use by)
	In vitro diagnostic medical device
	Catalog number
	CE Marking
	Caution, consult instructions for use
	Storage instructions
	Authorized Representative
	Manufacturer
	Lot number

### Procedure:


**Most people take between 2 and 5 minutes to deliver a sample following steps 1 to 4.**

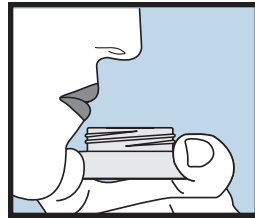


**1** Start giving your saliva sample.

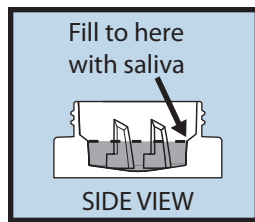

**2** Spit until the amount of liquid saliva (not bubbles) reaches the level shown in picture #2.

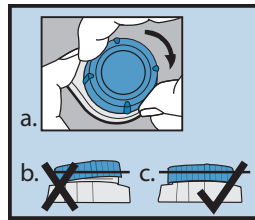

**3** Put the container on a flat surface. Screw the cap onto the container. Make sure that the cap is closed tightly, as shown in picture #3c. If you have difficulty closing the cap completely, turn cap slightly counter-clockwise then try again.

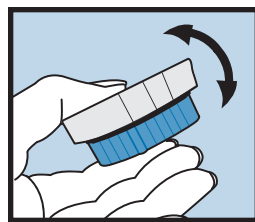

**4** Shake gently for 10 seconds. Recycle outer packaging.



For In Vitro Diagnostic Use

**DNAgenotek**



Made in Canada  
DNA Genotek Inc.  
3000 - 500 Palladium Drive  
Ottawa, ON, Canada K2V 1C2

*Superior samples*  
*Proven performance*

Toll-free (North America): 1.866.813.6354  
Tel: +1.613.723.5757 • Fax: +1.613.723.5057  
info@dnagenotek.com  
www.dnagenotek.com



Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

Oragene-DNA is not available for sale in the United States.

\*Oragene is a registered trademark of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions, contact your sales representative for details.

All DNA Genotek protocols, white papers and application notes, are available in the support section of our website at [www.dnagenotek.com](http://www.dnagenotek.com).

Patent ([www.dnagenotek.com/legalnotices](http://www.dnagenotek.com/legalnotices))

© 2018 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

PD-PR-129 Issue 7/2018-08