
Collection precautions:

- Do NOT remove the plastic film from the funnel lid.
- Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.
- Do NOT substitute with other sponges or swabs.

Intended use: For the assisted collection of human DNA from saliva samples.

Contents: Kit contains stabilizing liquid.

Warnings and precautions:
Choking hazards:

- Small cap in collection kit.
- Plastic bag containing sponges.
- Caution should be used when inserting sponge into donor's mouth.

For supervised collections:











- Do NOT leave donor unattended.
- Do NOT allow donor to handle the sponge, small cap or packaging.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do not ingest. See MSDS at www.dnagenotek.com.

Storage: 15°C \leftrightarrow 30°C

Summary and explanation of the kit:

Oragene-DNA is an assisted collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Label legend:

	Consult package insert
	Collect saliva by (Use by)
	In vitro diagnostic medical device
	Catalog number
	CE Marking
	Caution, consult instructions for use
	Storage instructions
	Authorized Representative
	Manufacturer
	Lot number

USER INSTRUCTIONS


Read all instructions prior to collection

Procedure:


Ensure donor does NOT eat, drink, smoke or chew gum for 30 minutes before collecting a saliva sample.

Ensure donor is in an upright position during sample collection.


It may take up to 15 minutes to collect a saliva sample following steps 1 to 7.

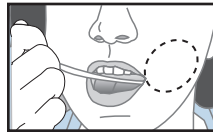


- Place one sponge in cheek pouch. Gently move the sponge along the gums and inner cheeks for 30 seconds to soak up as much saliva as possible.

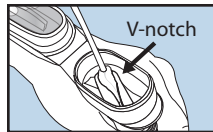


- Once sponge is saturated with saliva, insert sponge in V-notch of funnel. Wring saliva out of sponge using a twisting and pushing motion against the inner wall of the V-notch. Saliva will flow into tube.

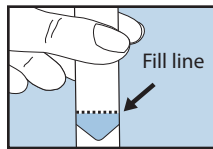


- Repeat these steps (1 to 2) USING THE SAME SPONGE until the liquid saliva (not bubbles) reaches the fill line. Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.
Tap tube bottom against hard surface to reduce number of bubbles.

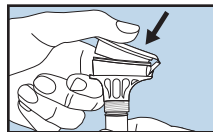

- Hold the tube upright with one hand. Close the lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure the lid is closed tightly.

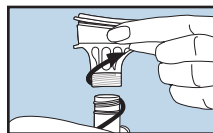

- Hold the tube upright. Unscrew the funnel from the tube.

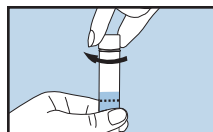

- Use the small cap to close the tube tightly.



- Shake the capped tube for 5 seconds. Discard the funnel and sponges.



For In Vitro Diagnostic Use

DNAgenotek



Made in Canada
 DNA Genotek Inc.
 3000 - 500 Palladium Drive
 Ottawa, ON, Canada K2V 1C2

Superior samples
Proven performance

Toll-free (North America): 1.866.813.6354
 Tel: +1.613.723.5757 • Fax: +1.613.723.5057
 info@dnagenotek.com
 www.dnagenotek.com



Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, The Netherlands

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

Oragene-DNA is not available for sale in the United States.

*Oragene is a registered trademark of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions, contact your sales representative for details.

All DNA Genotek protocols, white papers and application notes, are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)

© 2017 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

PD-PR-134 Issue 5/2017-05