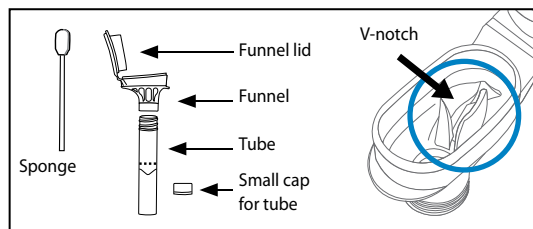
English – User instructions	2
Français – Mode d'emploi	3
Español – Instrucciones de usuario	4
Deutsch – Benutzeranleitung	5
Norsk – Bruksanvisning	6
Svenska – Bruksanvisning	7
Dansk – Brugsanvisninger	8
Nederlands – Gebruikersinstructies	9
Česky – Pokyny pro uživatele	10
Italiano – Istruzioni per l'uso	11
Polski – Instrukcje dla użytkownika	12
Português – Instruções do utilizador	13
Português – Brasil – Instruções ao usuário	14
العربية – تعليمات المستخدم	15
中文 – 用户须知	16
한글 – 사용 지침	17
日本語 – ユーザー指示書	18
Български – Инструкции за потребителя	19
Hrvatski – Upute za korisnike	20
Eesti keel – Kasutusjuhend	21
Suomi – Käyttöohjeet	22
Ελληνικά – Οδηγίες χρήσης	23
Magyar – Használati utasítás	24
Gaeilge – Treoracha úsáideora	25
Latviešu valoda – Lietotāja instrukcijas	26
Lietuvių k. – Naudotojo instrukcijos	27
Malti – Istruzzjonijiet tal-Utent	28
Română – Instrucțiuni de utilizare	29
Русский – Инструкция для пользователя	30
Slovensky – Návod na použitie	31
Slovenščina – Navodila za uporabnika	32

Collection precautions:

Do NOT remove the plastic film from the funnel lid.

Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.

Do NOT substitute with other sponges or swabs.

Intended use: For the assisted collection of human DNA from saliva samples

Contents: 1 kit containing 0.75 mL stabilizing liquid

Warnings and precautions:

Choking hazards:

- Small cap in collection kit.
- Plastic bag containing sponges.
- Caution should be used when inserting sponge into donor's mouth.

For supervised collections:

- Do NOT leave donor unattended.
- Do NOT allow donor to handle the sponge, small cap or packaging.
- Wash with water if stabilizing liquid comes in contact with eyes or skin.
- Do not ingest. See SDS at www.dnagenotek.com.

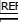
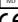
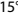
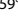
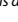
Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C / 30°C (59°F / 86°F)

Summary and explanation of the kit:

Oragene-DNA is an assisted collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Label legend:

	Flammable liquid
	Catalog number
	In vitro diagnostic medical device**
	CE marking
	UKCA marking
	Manufacturer
	Medical device*
	Do not reuse
	Consult instructions for use
	Collect saliva by (Use by)
	Lot number
	European Authorized Representative
	Swiss Authorized Representative
	Caution
	Unique Device Identifier
	Storage instructions

Per applicable regulatory requirements, the device contains a swab component, which is a *medical device, and a collection tube component, which is an **in vitro device.

User instructions

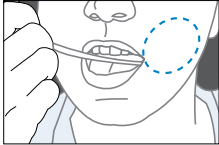
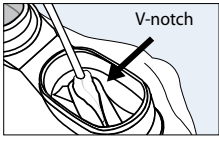
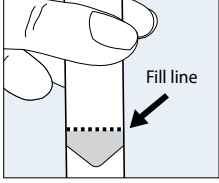
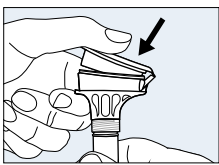
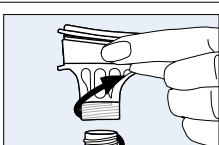
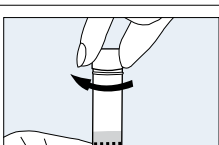
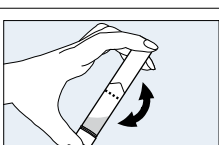
Read all instructions prior to collection.

Procedure:

Ensure donor does NOT eat, drink, smoke or chew gum for 30 minutes before collecting a saliva sample.

Ensure donor is in an upright position during sample collection.

It may take up to 15 minutes to collect a saliva sample following steps 1 to 7.

	1 Place one sponge in cheek pouch. Gently move the sponge along the gums and inner cheeks for 30 seconds to soak up as much saliva as possible.
	2 Once sponge is saturated with saliva, insert sponge in V-notch of funnel. Wring saliva out of sponge using a twisting and pushing motion against the inner wall of the V-notch. Saliva will flow into tube.
	3 Repeat these steps (1 to 2) USING THE SAME SPONGE until the liquid saliva (not bubbles) reaches the fill line. Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear. Tap tube bottom against hard surface to reduce number of bubbles.
	4 Hold the tube upright with one hand. Close the lid with the other hand by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure the lid is closed tightly.
	5 Hold the tube upright. Unscrew the funnel from the tube.
	6 Use the small cap to close the tube tightly.
	7 Shake the capped tube for 5 seconds. Discard the funnel and sponges.

Warning

CE MD UKCA

For In Vitro Diagnostic Use

DNAGENOTEK™

*Superior samples
Proven performance*

Made in Canada
DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

Novosanis NV, Bijkhoevelaan 32c, 2110 Wijnegem, Belgium Email: EUAR@novosanis.com

UK Responsible Person: Emergo Consulting (UK) Limited c/o Cr360 – UL International, Compass House, Vision Park Histon, Cambridge, CB24 9BZ

Arazy Group Swiss GmbH, Bruderholzallee 53, 4059 Basel, Schweiz Email: swiss.ar@arazygroup.com

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

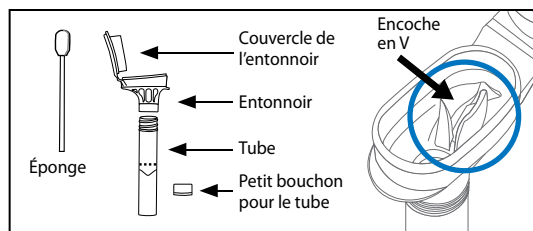
Oragene-DNA is not available for sale in the United States.

Oragene and DNA Genotek are trademarks of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legal/notice)

© 2024 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved. PD-PR-134 Issue 8/2024-04

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CH REF Arazy Group Swiss GmbH, Bruderholzallee 53, 4059 Bazilej, Švajčiarsko E-mail: swiss.ar@arazygroup.com
© 2024 DNA Genotek Inc., dcérska spoločnosť OraSure Technologies, Inc., všetky práva vyhradené.