**Read all instructions prior to collection.**

**For In Vitro Diagnostic Use**

**Collection precautions:**  
Ensure the sponge tip does NOT come into contact with any surface prior to collection.  
Donor should NOT eat, drink, smoke, or chew gum for 30 minutes before collecting saliva sample.

**Intended use:** For the collection and stabilization of RNA from human saliva samples

**Contents:** 1 kit containing 1 mL stabilizing liquid

**Warnings and precautions:**  
Choking hazard. Caution should be used when inserting sponge into the mouth.

Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.

Report any serious incident to DNA Genotek and the competent authority in your country.

**Storage:** 15°C (59°F) to 25°C (77°F)

**Summary and explanation of the kit:**  
ORACollect™ RNA is a self-collection kit that provides the materials and instructions for collecting human saliva samples.

**Instructions for sample collection:**

- Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.
- Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.
- Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the cap from the collection tube without touching the sponge.
- Turn the cap upside down, insert the sponge into the tube and close cap tightly.
- Invert the capped tube and shake vigorously 15 times.

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**DNagenotek™**

**REF** ORE-100  
DNA Genotek Inc.  
3000 - 500 Palladium Drive  
Ottawa, ON, Canada K2V 1C2  
Toll-free (North America):  
1.866.813.6354  
+1.613.723.5757  
info@dnagenotek.com  
www.dnagenotek.com

Patent (EN) Brevet (FR) Patente (ES)  
Patent (DE) Patent (NB) Patent (SV)  
Patent (DA) Patent (NL) Patent (CS)

**IVD**

**Label legend:**

- Rx ONLY** For prescription use only in the USA under EUA
- REF** Catalog number
- IVD** In vitro diagnostic medical device
- MA** Manufacturer
- RE** Do not reuse
- UI** Consult instructions for use
- CA** Caution
- SI** Use by
- LOT** Lot number
- UDI** Unique Device Identifier
- QTY** Quantity of units per package
- 15°C (59°F) to 25°C (77°F)** Storage instructions
- MADE IN US FROM FOREIGN AND DOMESTIC MATERIALS** Made in US from foreign and domestic materials

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© 2024 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved. PD-PR-00772 v7/2024-07