
REF: OG-575/OG-675
 DNA Genotek Inc.
 3000 - 500 Palladium Drive
 Ottawa, ON, Canada K2V 1C2
 Tel.: +1.613.723.5757
 info@dnagenotek.com
 www.dnagenotek.com

Patent (www.dnagenotek.com/legalnotices)
 PD-PR-00942 Issue 3/2023-12

Warning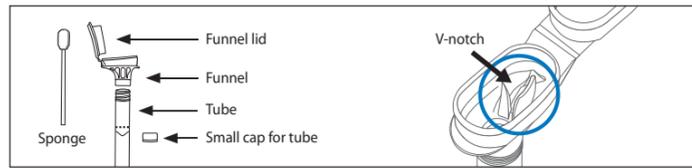

For In Vitro Diagnostic Use

Read all instructions prior to collection.

Intended use: For the assisted collection of human DNA from saliva samples

Contents: 1 kit containing 0.75 mL stabilizing liquid

Warnings and precautions:

- Choking hazards:
 - Small cap in collection kit.
 - Plastic bag containing sponges.
 - Caution should be used when inserting sponge into donor's mouth.
- For supervised collections:
 - Do NOT leave donor unattended.
 - Do NOT allow donor to handle the sponge, small cap or packaging.
 - Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15 °C | 30 °C (59 °F | 86 °F)

Summary and explanation of the kit:

Oragene®-DNA is an assisted collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Collection precautions:

- Do NOT remove the plastic film from the funnel lid.
- Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.
- Do NOT substitute with other sponges or swabs.

Superior samples • Proven performance

Novosanis NV, Bijkhoevelaan 32c, 2110 Wijnegem, Belgium
 Email: EUAR@novosanis.com

UK Responsible Person: Emergo Consulting (UK) Limited c/o Cr360 - UL International, Compass House, Vision Park Histon, Cambridge, CB24 9BZ

Arazy Group Swiss GmbH
 Bruderholzallee 53, 4059 Basel, Schweiz
 Email: swiss.ar@arazygroup.com


Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

© 2023 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

Label legend:

- Flammable liquid
- Catalog number
- In vitro diagnostic medical device**
- CE marking
- UKCA marking
- Manufacturer
- Medical device*
- Do not reuse
- Consult instructions for use
- Collect saliva by (Use by)
- Lot number
- European Authorized Representative
- Swiss Authorized Representative
- Caution
- Unique Device Identifier
- 15 °C | 30 °C Storage instructions
- 59 °F | 86 °F

Per applicable regulatory requirements, the device contains a swab component, which is a *medical device, and a collection tube component, which is an **in vitro device.

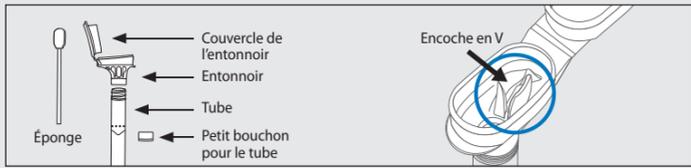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© 2023 DNA Genetek Inc., daughter company of OraSure Technologies, Inc.; all rights reserved.