




REF OGD-500 and OGD-510

REF OGD-600 and OGD-610

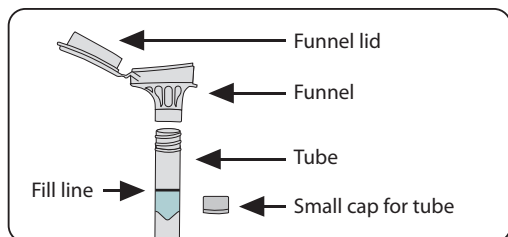

Intended use/purpose: For the collection of human DNA from saliva samples.

Intended user: This product is intended for use in adults.

Contents: 1 kit containing stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com.

Small cap may pose a choking hazard.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C / 30°C (59°F / 86°F)

Summary and explanation of the kit:

Oragene®-Dx is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Collection precautions:

Do NOT eat, drink, smoke, or chew gum for 30 minutes before giving your saliva sample.

Do NOT remove the plastic film from the funnel lid.

Label legend:


	Flammable liquid
	Catalog number
	CE marking
	In vitro diagnostic medical device
	UKCA marking
	Do not re-use
	Consult package insert
	Caution, consult instructions for use
	Manufacturer
	European Authorized Representative
	Swiss Authorized Representative
	Collect saliva by (Use by)
	Lot number
	Unique Device Identifier
15 °C / 30 °C	Storage instructions
59 °F / 86 °F	

USER INSTRUCTIONS


Read all instructions prior to collection.

Procedure:


Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 5.

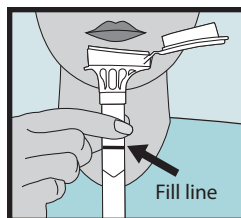


- 1 Spit into funnel until the amount of liquid saliva (not bubbles) reaches the Fill line shown in picture #1.

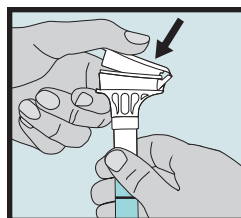


- 2 Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure that the lid is closed tightly.

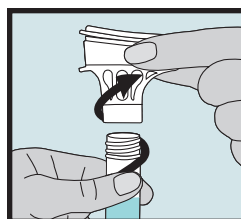

- 3 Hold the tube upright. Unscrew the funnel from the tube.

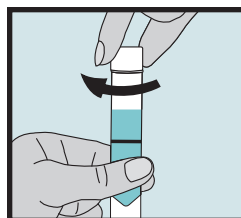

- 4 Use the small cap to close the tube tightly.

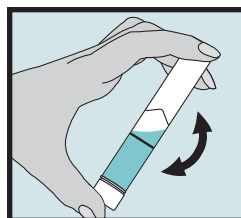

- 5 Shake the capped tube for 5 seconds. Discard or recycle the funnel.

See additional instructions for use PD-HB-00001 at www.dnagenotek.com.



For In Vitro Diagnostic Use

For Prescription and Direct-to-Consumer Use

DNagenotek™

*Superior samples
Proven performance*

Made in Canada
DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

UK Responsible Person: Emergo Consulting (UK) Limited c/o Cr360 - UL International, Compass House, Vision Park Histon, Cambridge, CB24 9BZ

EC REP Novosanis NV Bijlkhoevelaan 32c, 2110 Wijnegem, Belgium Email: EUA@novosanis.com

CH REP Arazy Group Swiss GmbH, Bruderholzallee 53, 4059 Basel, Schweiz Email: swiss.ar@arazygroup.com

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

Oragene is a registered trademark and DNA Genotek is a trademark of DNA Genotek Inc. Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)

© 2022 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.
PD-PR-00429 Issue 6/2022-05