EN English – Procedure for saliva sample collection.....	2
FR Français – Procédure de prélèvement d'un échantillon de salive	3
ES Español – Procedimiento para la recogida de la muestra de saliva	4
DE Deutsch – Verfahren zur Entnahme einer Speichelprobe	5
NB Norsk – Prosedyre for innsamling av spyttprøve	6
SV Svenska – Procedur för insamling av salivprov	7
DA Dansk – Procedure til indsamling af spyttprøve	8
NL Nederlands – Procedure voor afname van speekselmonster	9
CS Česky – Postup odběru vzorku slin	10
IT Italiano – Procedura per la raccolta del campione di saliva	11
PL Polski – Procedura pobierania próbek śliny	12
PT Português – Procedimento para a recolha de amostras de saliva	13
BZ Português – Brasil – Procedimento para a recolha de amostras de saliva	14
AR العربية – إجراء جمع عينة اللعاب	15
ZH 中文 – 唾液样本采集步骤	16
KO 한글 – 타액 샘플 채취용 절차	17
JA 日本語 – 唾液のサンプル採取方法	18
BG Български – Процедура за вземане на проба от слюнка	19
HR Hrvatski – Postupak uzimanja uzorka pljuvačke	20
ET Eesti keel – Süljeproovi võtmise protseduur	21
FI Suomi – Sylkinäytteen ottaminen	22
EL Ελληνικά – Διαδικασία λήψης δείγματος σιέλου	23
HU Magyar – Nyálminták gyűjtésének eljárása	24
IE Gaeilge – Gnáthamh maidir le bailiú samplaí seile	25
LV Latviešu valoda – Procedūra siekalu parauga paņemšanai	26
LT Lietuvių k. – Seilių mėginio paėmimo procedūra	27
MT Malti – Procedura għall-gbir tal-bżieq	28
RO Română – Procedura de colectare a probelor de salivă	29
RU Русский – Процедура сбора образца слюны	30
SK Slovensky – Postup na odber vzorky slín	31
SL Slovenščina – Postopek za jemanje vzorca sline	32
TH עברית – הליך לאיסוף דגימת רוק	33

Collection precautions:

Donor should NOT eat, drink, smoke, or chew gum for 30 minutes before collecting saliva sample.

Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Intended use: ORAc collect™ Dx is intended for the collection of saliva samples for diagnostic testing of human DNA. Saliva samples may be collected by a healthcare professional or non-healthcare professional, such as a lay user. Saliva samples collected using ORAc collect™ Dx are stabilized and isolated for use in downstream diagnostic testing applications. Saliva samples collected using ORAc collect™ Dx can be transported and/or stored at ambient conditions.

Contents: 1 kit containing 1 mL stabilizing liquid.

Warnings and precautions:

- Read all instructions carefully prior to sample collection; deviation may result in inadequate sample and impact DNA yield.
- For use in individuals 18 years of age and older.
- Choking hazard. Caution should be used when inserting sponge into the mouth.
- Donors with xerostomia (dry mouth) may not collect adequate sample using these instructions, resulting in lower DNA yield and an invalid sample for use.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.

Report any serious incident to DNA Genotek and the competent authority in your country.

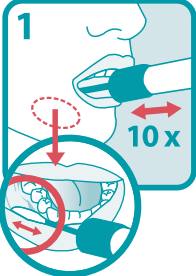
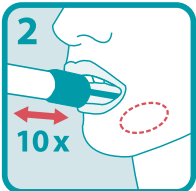
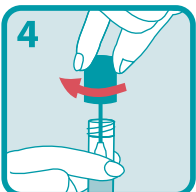
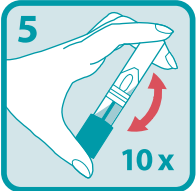
Storage: 15°C / 25°C (59°F / 77°F)

Label legend:

[REF]	Catalog number
[IVD]	In vitro diagnostic medical device
[M]	Manufacturer
[2]	Do not re-use
[i]	Consult instructions for use
[A]	Caution
[U]	Collect sample by (Use by)
[LOT]	Lot number
[UDI]	Unique Device Identifier
[QTY]	Quantity of units per package
15°C / 25°C	Storage instructions
59°F / 77°F	
MADE IN US FROM FOREIGN AND DOMESTIC MATERIALS	Made in US from foreign and domestic materials

ENGLISH

Procedure for saliva sample collection:

	<p>Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.</p>
	<p>Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.</p>
	<p>Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.</p>
	<p>Turn the cap upside down, insert the sponge into the tube and close cap tightly.</p>
	<p>Invert the capped tube and shake vigorously 10 times.</p>

[IVD]

For In Vitro Diagnostic Use

DNAGENOTEK™

*Superior samples
Proven performance*

DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

ORAc collect and DNA Genotek are trademarks of DNA Genotek Inc., registered under various jurisdictions. Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)
© 2025 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.
PD-PR-506 v10/2025-01

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DNAGENOTEK Inc.