
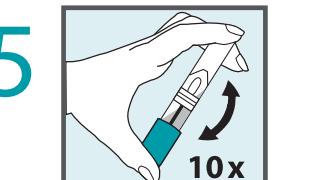
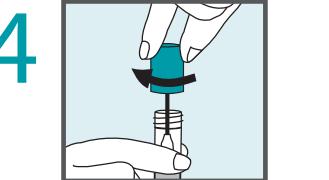
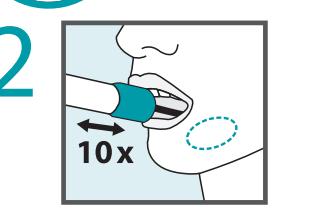
ORACollect™-Dx

ENGLISH User instructions

Read all instructions prior to collection.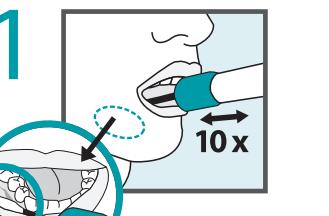

DNAgenotek™

REF OCD-100A
DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2
Toll-free (North America):
1.866.813.6354
+1.613.723.5757
info@dnagenotek.com
www.dnagenotek.com

Patent (EN) Brevet (FR) Patente (ES)
Patent (DE) Patent (NB) Patent (SV)
Patent (DA) Patent (NL) Patent (CS)
(www.dnagenotek.com/legalnotices)

IVD

Label legend:

REF	Catalog number
IVD	In vitro diagnostic medical device
■	Manufacturer
⊗	Do not re-use
□	Consult the mode d'emploi
Caution	
Collect sample by (Use by)	
Lot number	
Unique Device Identifier	
Quantity of units per package	
15°C / 25°C	Instructions of conservation
59°F / 77°F	
Made in US from foreign and domestic materials	

Legend:

REF	Número de referencia
IVD	Dispositif médical pour diagnostic in vitro
■	Fabricant
⊗	Ne pas réutiliser
□	Consulter le mode d'emploi
Caution	
Collect sample by (Use by)	
Lot number	
Unique Device Identifier	
Quantity of units per package	
15°C / 25°C	Instructions de conservation
59°F / 77°F	
Made in US from foreign and domestic materials	

Made in US from foreign and domestic materials

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For In Vitro Diagnostic Use

Collection precautions:
Donor should NOT eat, drink, smoke, or chew gum for 30 minutes before collecting saliva sample.
Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Intended use: ORACollect™-Dx is intended for the collection of saliva samples for diagnostic testing of human DNA. Saliva samples may be collected by a healthcare professional or non-healthcare professional, such as a lay user. Saliva samples collected using ORACollect™-Dx are stabilized and isolated for use in downstream diagnostic testing applications. Saliva samples collected using ORACollect™-Dx can be transported and/or stored at ambient conditions.

Contents: 1 kit containing 1 mL stabilizing liquid

Warnings and precautions:

- Read all instructions carefully prior to sample collection; deviation may result in inadequate sample and impact DNA yield.
- For use in individuals 18 years of age and older.
- Choking hazard. Caution should be used when inserting sponge into the mouth.
- Donors with xerostomia (dry mouth) may not collect adequate sample using these instructions, resulting in lower DNA yield and an invalid sample for use.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.
- Report any serious incident to DNA Genotek and the competent authority in your country.
- **Storage:** 15°C / 25°C (59°F / 77°F)

Procedure for saliva sample collection:

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