

**Intended use/purpose:** Intended for the collection and stabilization of microbial DNA from human fecal samples.

**Intended user:** This product is intended for use by adults.

**Contents:** 1 kit containing stabilizing liquid.

**Summary and explanation of the kit:**











OMNIGene<sup>®</sup>GUT provides the materials and instructions for collecting and stabilizing microbial DNA from a fecal sample.

**Warnings and precautions:**

- FOR EXTERNAL USE ONLY.
- Do NOT remove the yellow tube top from the tube.
- Do NOT spill the stabilizing liquid in the tube.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at [www.dnagenotek.com](http://www.dnagenotek.com).
- If collecting a liquid fecal sample, see separately provided user instructions.
- Small items may pose a choking hazard.
- Report any serious incident to DNA Genotek and the competent authority in your country

**Storage:** Room temperature storage pre- and post-collection 15 °C / 25 °C (59 °F / 77 °F).

**Label legend:**

-  Flammable liquid
  -  Exclamation mark (Warning)
  -  Manufacturer
  -  Collect sample by (Use by)
  -  Catalog number
  -  Lot number
  -  Do not re-use
  -  Consult package insert
  -  Caution, consult instructions for use
  -  Storage instructions
- 15°C / 25°C  
59°F / 77°F

## USER INSTRUCTIONS

Read all instructions prior to collection.

Other languages are available at [www.dnagenotek.com](http://www.dnagenotek.com)


**Procedure:**





- 1 IMPORTANT PREPARATIONS:**
- Empty your bladder before beginning the collection.
  - Collect fecal sample free of urine or toilet water.
  - Utilize any toilet accessory which may be supplied with the collection device.
  - Toilet paper or tissues may be required.




- 2** While holding the yellow tube top, unscrew ONLY the purple cap and set aside for later use.
- IMPORTANT:** Do NOT remove the yellow tube top. Tube is not designed to stand upright. Do NOT spill the stabilizing liquid in the tube.

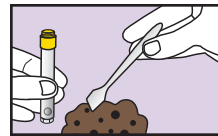


- 3** Use the spatula to collect a small amount of fecal sample.

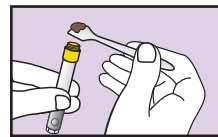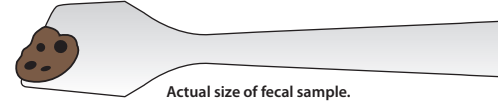


- 4** Transfer the fecal sample into the top portion of the yellow tube top. Repeat until the sample fills the yellow tube top.

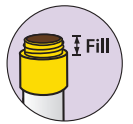


**IMPORTANT:** Do NOT push sample into the tube.

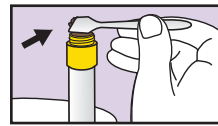


- 5** Scrape horizontally across the tube top to level the sample and remove any excess.
- Wipe exterior of tube and top with toilet paper or tissue as needed.

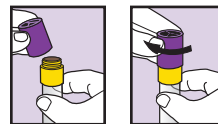

- 6** Pick up the purple cap and screw onto the yellow tube top until tightly closed.

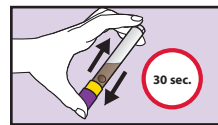

- 7** For a **minimum of 30 seconds**, shake the sealed tube as hard and fast as possible in a back and forth motion.

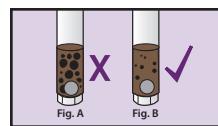

- 8** The fecal sample will be mixed with the stabilizing liquid in the tube; not all particles will dissolve.
- IMPORTANT:** Continue shaking if **large** particles remain as shown in Figure A.



- 9** Place spatula in original packaging or wrap in toilet paper and discard in garbage.
- IMPORTANT:** Send the sample for processing following the delivery instructions supplied separately by the kit provider.

Ship in accordance to applicable regulations covering transport of biological specimens.

**For Research Use Only**  
Not for use in diagnostic procedures

**DNAGENOTEK™**

*Superior samples*  
*Proven performance*

Made in Canada  
DNA Genotek Inc.  
3000 - 500 Palladium Drive  
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354  
Tel: +1.613.723.5757 • Fax: +1.613.723.5057  
[info.dnagenotek.com](http://info.dnagenotek.com)  
[www.dnagenotek.com](http://www.dnagenotek.com)

OMNIGene-GUT (OMR-200) is for research use only, not for use in diagnostic procedures. OMNIGene is a registered trademark and DNA Genotek is a trademark of DNA Genotek Inc. Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at [www.dnagenotek.com](http://www.dnagenotek.com).

Patent ([www.dnagenotek.com/legalnotices](http://www.dnagenotek.com/legalnotices))  
© 2022 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.  
PD-PR-01274 Issue 1/2022-05