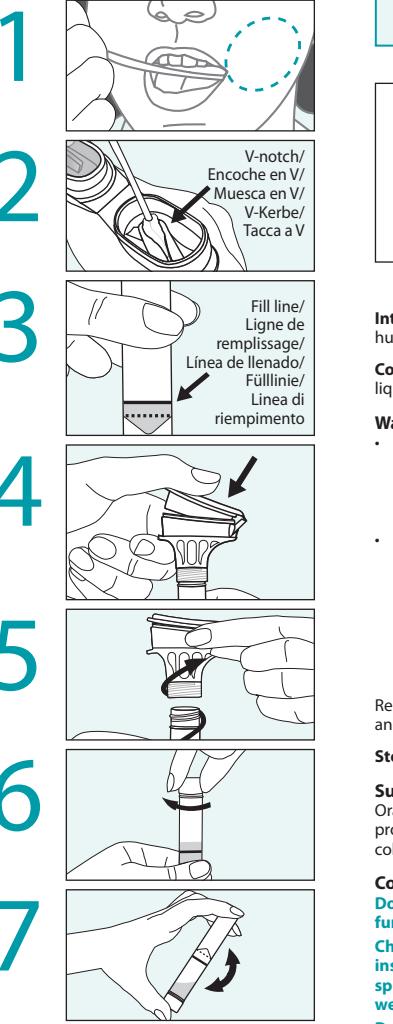
ORAGene™-Dx

REF OGD-575, REF OGD-675

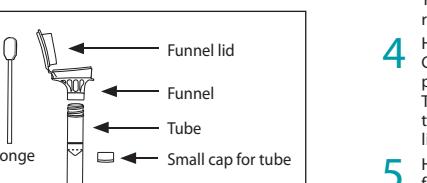
DNAgenotek™

DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America):
1.866.813.6354
+1.613.723.5757
info@dnagenotek.com
www.dnagenotek.com

CE IVD UK CA

EN ENGLISH – User instructions
Read all instructions prior to collection.
For In Vitro Diagnostic Use

Intended use: For the assisted collection of human DNA from saliva samples.

Contents: 1 kit containing 0.75 mL stabilizing liquid.

Warnings and precautions:

- Choking hazards:
 - Small cap in collection kit.
 - Plastic bag containing sponges.
 - Caution should be used when inserting sponge into donor's mouth.
- For supervised collections:
 - Do NOT leave donor unattended.
 - Do NOT allow donor to handle the sponge, small cap or packaging.
 - Wash with water if stabilizing liquid comes in contact with eyes or skin.
 - Do NOT ingest. See SDS at www.dnagenotek.com.

Report any serious incident to DNA Genotek and the competent authority in your country.

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Summary and explanation of the kit:
Oragene™-Dx is an assisted collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Collection precautions:

- Do NOT remove the plastic film from the funnel lid.**
Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.
Do NOT substitute with other sponges or swabs.

Procedure:

- Ensure donor does NOT eat, drink, smoke, or chew gum for 30 minutes before collecting a saliva sample.
- Ensure donor is in an upright position during sample collection.
- It may take up to 15 minutes to collect a saliva sample following steps 1 to 7.

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1 Place one sponge in cheek pouch. Gently move the sponge along the gums and inner cheeks for 30 seconds to soak up as much saliva as possible.

2 Once sponge is saturated with saliva, insert sponge in V-notch of funnel. Wring saliva out of sponge using a twisting and pushing motion against the inner wall of the V-notch. Saliva will flow into tube.

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- 3** Repeat these steps (1 to 2) USING THE SAME SPONGE until the liquid saliva (not bubbles) reaches the fill line. Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear. Tap tube bottom against hard surface to reduce number of bubbles.
- 4** Hold the tube upright with one hand. Close the lid with the other hand by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure the lid is closed tightly.

- 5** Hold the tube upright. Unscrew the funnel from the tube.
- 6** Use the small cap to close the tube tightly.

- 7** Shake the capped tube for 5 seconds. Discard the funnel and sponges.

See instructions for use (PD-PR-430) at www.dnagenotek.com.

Label legend:

REF	Flammable liquid
IVD	Catalog number
CE	In vitro diagnostic medical device
UKCA	CE marking
REF	UKCA marking
IVD	Manufacturer
CE	Do not reuse
UKCA	Consult instructions for use
REF	Collect saliva by (Use by)
IVD	Lot number
CE	European Authorized Representative
UKCA	Swiss Authorized Representative
REF	Caution
IVD	Unique Device Identifier
CE	Quantity of units per package
UKCA	Storage instructions
REF	Made in US from foreign and domestic materials
IVD	Brevet (www.dnagenotek.com/legalnotices)
CE	© 2024 DNA Genotek Inc., une filiale d'OraSure Technologies, Inc., tous droits réservés.
UKCA	

Made in US from foreign and domestic materials

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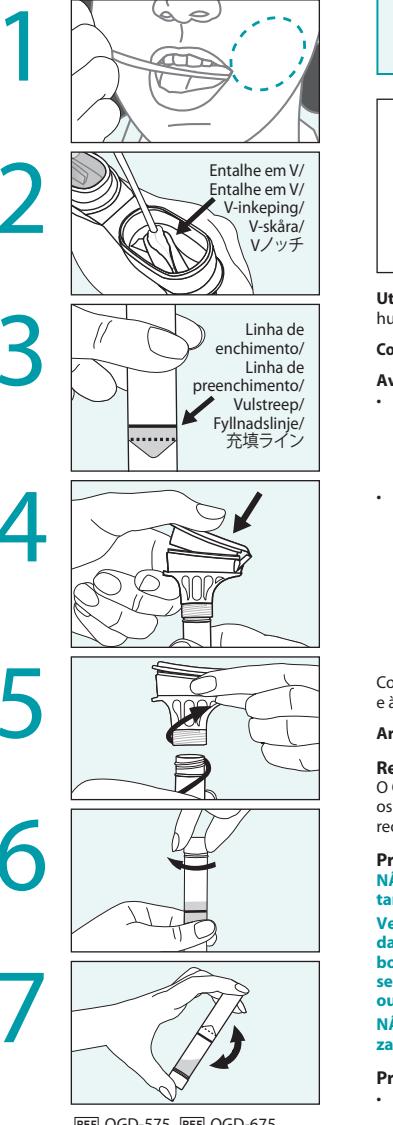
<p

REF OGD-575, REF OGD-675

EC REP Qarad EC-REP BV, Pas 257, 2440 Geel, Belgium
UK Responsible Person: Qarad UK Ltd.,
8 Northumberland Ave, Westminster, London,
WC2N 5BY United KingdomCH REP Qarad Suisse S.A., World Trade Center,
Avenue Gratta-Paille 2, 1018 Lausanne, SwitzerlandAustralian Sponsor: Emergo Australia, Level 20,
Tower II, Darling Park, 201 Sussex Street, Sydney© 2024 DNA Genotek Inc., a subsidiary of
OraSure Technologies, Inc., all rights reserved.
PD-PR-00941 v5/2024-07

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