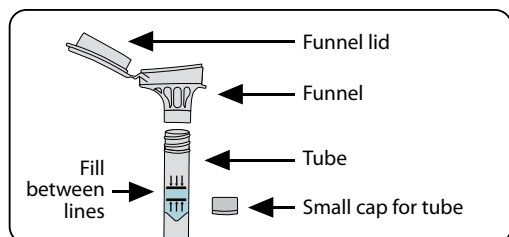
EN English – User instructions	2
FR Français – Mode d'emploi	3
ES Español – Instrucciones de uso	4
DE Deutsch – Gebrauchsanweisung	5
NB Norsk – Bruksanvisning	6
SV Svenska – Användarinstruktioner	7
DA Dansk – Brugsanvisning	8
NL Nederlands – Gebruiksaanwijzing	9
CS Český – Pokyny pro uživatele	10
IT Italiano – Istruzioni per l'utente	11
PL Polski – Instrukcja obsługi	12
PT Português – Instruções do utilizador	13
BZ Português (Brasil) – Instruções de uso	14
AR العربية – تعليمات المستخدم	15
ZH 中文 – 使用说明	17
KO 한글 – 사용 설명서	18
JA 日本語 – 取扱説明書	19
BG Български – Инструкции за потребителя	20
HR Hrvatski – Korisničke upute	21
ET Eesti keel – Kasutusjuhend	22
FI Suomi – Käyttöohjeet	23
EL Ελληνικά – Οδηγίες χρήστη	24
HU Magyar – Használati utasítás	25
IE Gaeilge – Treoracha úsáideora	26
LV Latviešu valoda – Lietotāja pamācība	27
LT Lietuvių k. – Naudotojo instrukcijos	28
MT Malti – Istruzzjonijiet għall-utent	29
RO Română – Instrucțiuni de utilizare	30
RU Русский – Инструкции для пользователя	31
SK Slovensky – Návod na použitie	32
SL Slovenščina – Navodila za uporabo	33

Intended use/purpose: For the collection of human DNA from saliva samples.

Intended user: This product is intended for use in adults.

Contents: 1 kit containing 1.5 mL stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.

Small cap may pose a choking hazard.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C / 30°C (59°F / 86°F)

Summary and explanation of the kit:

OrageneTM•ONE is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Collection precautions:

Do NOT eat, drink, smoke, or chew gum for 30 minutes before giving your saliva sample.

Do NOT remove the plastic film from the funnel lid.

Do NOT spit above the top line on the tube (picture 1).

Label legend:

	Flammable liquid
REF	Catalog number
CE	CE marking
IVD	In vitro diagnostic medical device
UK	UKCA marking
	Do not re-use
	Consult package insert
	Caution
	Manufacturer
EC REP	European Authorized Representative
CH REP	Swiss Authorized Representative
	Collect saliva by (Use by)
LOT	Lot number
UDI	Unique Device Identifier
QTY	Quantity of units per package
15 °C / 30 °C	Storage instructions
59 °F / 86 °F	

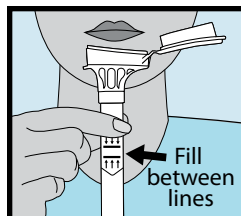
ENGLISH

USER INSTRUCTIONS


Read all instructions prior to collection.

Procedure:


Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 5.

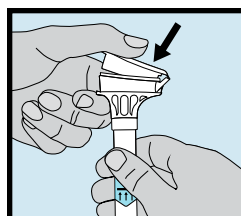
- 1 Spit a **SMALL** amount of saliva into the funnel. Gently tap the base of the tube on a flat surface to help the saliva move into the tube. Check that liquid saliva (not bubbles) level is between the lines as shown in picture #1.

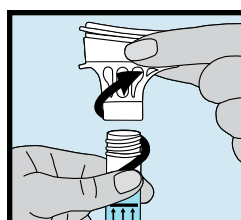
- 2 Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure that the lid is closed tightly.

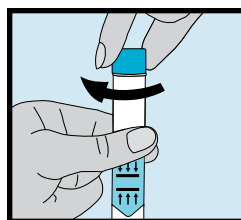- 3 Hold the tube upright. Unscrew the funnel from the tube.

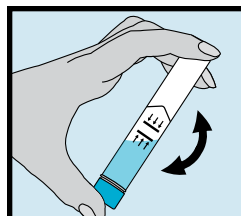- 4 Use the small cap to close the tube tightly.

- 5 Shake the capped tube for 5 seconds. Discard or recycle the funnel.

Warning

CE IVD UK

For In Vitro Diagnostic Use

DNAGENOTEKTM

*Superior samples
Proven performance*

DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

EC REP QbD RepS BV, Groenenborgerlaan 16, 2610 Wilrijk, Belgium

UK Responsible Person: Qarad UK Ltd., 8 Northumberland Ave, Westminster, London, WC2N 5BY United Kingdom

CH REP Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Switzerland

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

Oragene•ONE is not available for sale in the United States.

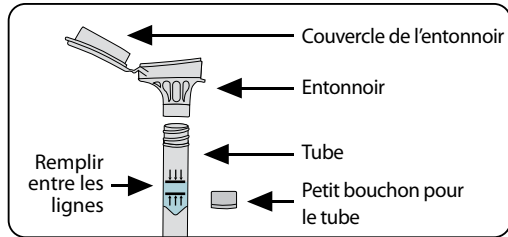
Oragene is a trademark of DNA Genotek Inc., registered in various jurisdictions.

Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legal-notice)

© 2025 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

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Odgovorna oseba v ZK: Qarad UK Ltd., 8 Northumberland Ave, Westminster, London, WC2N 5BY Združeno kraljestvo
CH REP Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Švica
OrAgene je blagovna znamka družbe DNA Genotek Inc., registrirana v različnih jurisdikcijah.
© 2025 DNA Genotek Inc., podružnica OraSure Technologies, Inc., vse pravice pridržane.