**ORAgene<sup>TM</sup>-Dx**

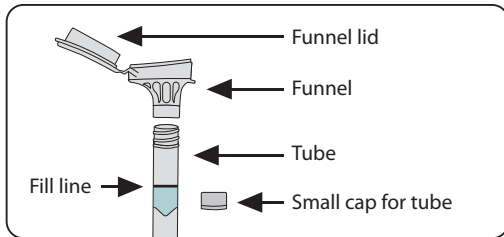
REF OGD-500, REF OGD-510

REF OGD-600, REF OGD-610

|    |                                              |    |
|----|----------------------------------------------|----|
| EN | English – User instructions .....            | 2  |
| FR | Français – Mode d’emploi .....               | 3  |
| ES | Español – Instrucciones de uso .....         | 4  |
| DE | Deutsch – Gebrauchsanweisung .....           | 5  |
| NB | Norsk – Bruksanvisning .....                 | 6  |
| SV | Svenska – Användarinstruktioner .....        | 7  |
| DA | Dansk – Brugsanvisning .....                 | 8  |
| NL | Nederlands – Gebruiksaanwijzing .....        | 9  |
| CS | Česky – Pokyny pro uživatele .....           | 10 |
| IT | Italiano – Istruzioni per l’utente .....     | 11 |
| PL | Polski – Instrukcja obsługi .....            | 12 |
| PT | Português – Instruções do utilizador .....   | 13 |
| BZ | Português (Brasil) – Instruções de uso ..... | 14 |
| AR | العربية – تعليمات المستخدم .....             | 15 |
| ZH | 中文 – 使用说明 .....                              | 16 |
| KO | 한글 – 사용 설명서 .....                            | 17 |
| JA | 日本語 – 取扱説明書 .....                            | 18 |
| BG | Български – Инструкции за потребителя .....  | 19 |
| HR | Hrvatski – Korisničke upute .....            | 20 |
| ET | Eesti keel – Kasutusjuhend .....             | 21 |
| FI | Suomi – Käyttöohjeet .....                   | 22 |
| EL | Ελληνικά – Οδηγίες χρήστη .....              | 23 |
| HU | Magyar – Használati utasítás .....           | 24 |
| IE | Gaeilge – Treoracha úsáideora .....          | 25 |
| LV | Latviešu valoda – Lietotāja pamācība .....   | 26 |
| LT | Lietuvių k. – Naudotojo instrukcijos .....   | 27 |
| MT | Malti – Istruzzjonijiet għall-utent .....    | 28 |
| RO | Română – Instrucțiuni de utilizare .....     | 29 |
| RU | Русский – Инструкции для пользователя .....  | 30 |
| SK | Slovensky – Návod na použitie .....          | 31 |
| SL | Slovenščina – Navodila za uporabo .....      | 32 |
| HE | עברית – הוראות למשתמש .....                  | 33 |

REF OGD-500, REF OGD-510  
REF OGD-600, REF OGD-610

**Intended use/purpose:** For the collection of human DNA from saliva samples.

**Intended user:** This product is intended for use in adults.

**Contents:** 1 kit containing 2 mL stabilizing liquid.

**Warnings and precautions:** Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at [www.dnagenotek.com](http://www.dnagenotek.com).

Small cap may pose a choking hazard.

Report any serious incident to DNA Genotek and the competent authority in your country.

**Storage:** 15°C / 30°C (59°F / 86°F)

### Summary and explanation of the kit:

Oragene™•Dx is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

### Collection precautions:

**Do NOT eat, drink, smoke, or chew gum for 30 minutes before giving your saliva sample.**

**Do NOT remove the plastic film from the funnel lid.**

### Label legend:

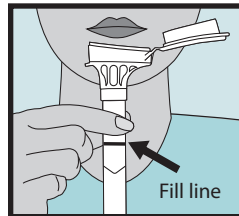
|               |                                    |
|---------------|------------------------------------|
|               | Flammable liquid                   |
| REF           | Catalog number                     |
| CE            | CE marking                         |
| IVD           | In vitro diagnostic medical device |
| UKCA          | UKCA marking                       |
|               | Do not re-use                      |
|               | Consult package insert             |
|               | Caution                            |
|               | Manufacturer                       |
| EC REP        | European Authorized Representative |
| CH REP        | Swiss Authorized Representative    |
|               | Collect saliva by (Use by)         |
| LOT           | Lot number                         |
| UDI           | Unique Device Identifier           |
| QTY           | Quantity of units per package      |
| 15 °C / 30 °C | Storage instructions               |
| 59 °F / 86 °F |                                    |

## USER INSTRUCTIONS


**Read all instructions prior to collection.**

### Procedure:


**Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 5.**

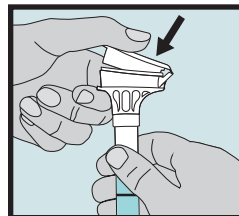
- 1** Spit into funnel until the amount of liquid saliva (not bubbles) reaches the Fill line shown in picture #1.

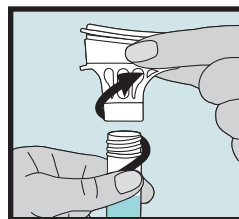
- 2** Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure that the lid is closed tightly.

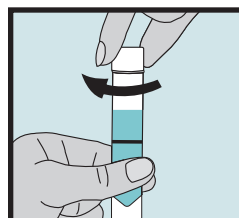- 3** Hold the tube upright. Unscrew the funnel from the tube.

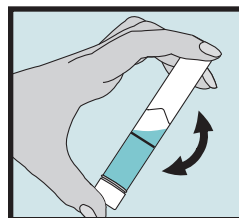- 4** Use the small cap to close the tube tightly.

- 5** Shake the capped tube for 5 seconds. Discard or recycle the funnel.

For U.S. distribution only, see the Oragene™•Dx product handbook DNA Genotek. PD-HB-1. [PDF]

Warning

CE IVD UKCA

For In Vitro Diagnostic Use

**DNAGENOTEK™**

*Superior samples  
Proven performance*

DNA Genotek Inc.  
3000 - 500 Palladium Drive  
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354  
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057  
[info@dnagenotek.com](mailto:info@dnagenotek.com)  
[www.dnagenotek.com](http://www.dnagenotek.com)

EC REP QbD RepS BV, Groenenborgerlaan 16, 2610 Wilrijk, Belgium

UK Responsible Person: Qarad UK Ltd., 8 Northumberland Ave, Westminster, London, WC2N 5BY United Kingdom

CH REP Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Switzerland

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

Oragene is a trademark of DNA Genotek Inc., registered in various jurisdictions.

Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at [www.dnagenotek.com](http://www.dnagenotek.com).

Patent ([www.dnagenotek.com/legal-notices](http://www.dnagenotek.com/legal-notices))

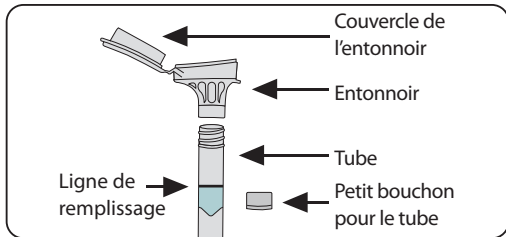
© 2025 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

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Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Switzerland [CH REF]