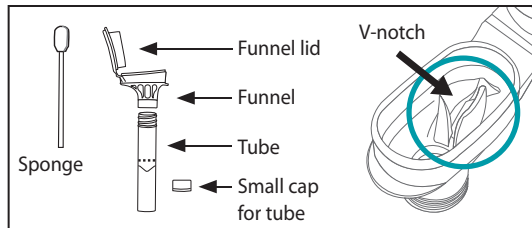
EN English – User instructions	2
FR Français – Mode d'emploi	3
ES Español – Instrucciones de usuario	4
DE Deutsch – Benutzeranleitung	5
NB Norsk – Bruksanvisning	6
SV Svenska – Bruksanvisning	7
DA Dansk – Brugsanvisninger	8
NL Nederlands – Gebruikersinstructies	9
CS Česky – Pokyny pro uživatele	10
IT Italiano – Istruzioni per l'uso	11
PL Polski – Instrukcje dla użytkownika	12
PT Português – Instruções do utilizador	13
BZ Português – Brasil – Instruções ao usuário	14
AR العربية – تعليمات المستخدم	15
ZH 中文 – 用戶須知	17
KO 한글 – 사용 지침	18
JA 日本語 – ユーザー指示書	19
BG Български – Инструкции за потребителя	20
HR Hrvatski – Upute za korisnike	21
ET Eesti keel – Kasutusjuhend	22
FI Suomi – Käyttöohjeet	23
EL Ελληνικά – Οδηγίες χρήσης	24
HU Magyar – Használati utasítás	25
IE Gaeilge – treoracha úsáideora	26
LV Latviešu valoda – Lietotāja instrukcijas	27
LT Lietuvių k. – Naudotojo instrukcijos	28
MT Malti – Istruzzjonijiet tal-Utent	29
RO Română – Instrucțiuni de utilizare	30
RU Русский – Инструкция для пользователя	31
SK Slovensky – Návod na použitie	32
SL Slovenščina – Navodila za uporabnika	33

Collection precautions:

Do NOT remove the plastic film from the funnel lid.

Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.

Do NOT substitute with other sponges or swabs.

Intended use: For the assisted collection of human DNA from saliva samples

Contents: 1 kit containing 0.75 mL stabilizing liquid

Warnings and precautions:

Choking hazards:

- Small cap in collection kit.
- Plastic bag containing sponges.
- Caution should be used when inserting sponge into donor's mouth.

For supervised collections:

- Do NOT leave donor unattended.
- Do NOT allow donor to handle the sponge, small cap or packaging.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do not ingest. See SDS at www.dnagenotek.com.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C / 30°C (59°F / 86°F)

Summary and explanation of the kit:

ORAgene™ Dx is an assisted collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Label legend:

	Flammable liquid
	Catalog number
	In vitro diagnostic medical device
	CE marking
	UKCA marking
	Manufacturer
	Do not reuse
	Consult instructions for use
	Collect saliva by (Use by)
	Lot number
	European Authorized Representative
	Swiss Authorized Representative
	Caution
	Unique Device Identifier
	Quantity of units per package
15 °C / 30 °C	Storage instructions
59 °F / 86 °F	

ENGLISH

User instructions

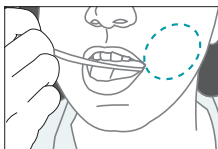
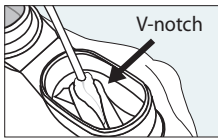
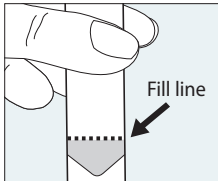
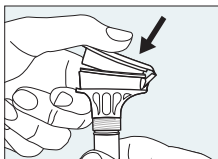
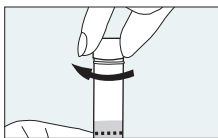
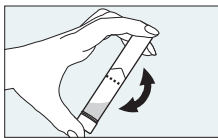
Read all instructions prior to collection.

Procedure:

Ensure donor does NOT eat, drink, smoke, or chew gum for 30 minutes before collecting a saliva sample.

Ensure donor is in an upright position during sample collection.

It may take up to 15 minutes to collect a saliva sample following steps 1 to 7.

	1 Place one sponge in cheek pouch. Gently move the sponge along the gums and inner cheeks for 30 seconds to soak up as much saliva as possible.
	2 Once sponge is saturated with saliva, insert sponge in V-notch of funnel. Wring saliva out of sponge using a twisting and pushing motion against the inner wall of the V-notch. Saliva will flow into tube.
	3 Repeat these steps (1 to 2) USING THE SAME SPONGE until the liquid saliva (not bubbles) reaches the fill line. Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear. Tap tube bottom against hard surface to reduce number of bubbles.
	4 Hold the tube upright with one hand. Close the lid with the other hand by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure the lid is closed tightly.
	5 Hold the tube upright. Unscrew the funnel from the tube.
	6 Use the small cap to close the tube tightly.
	7 Shake the capped tube for 5 seconds. Discard the funnel and sponges.

For U.S. distribution only, see the OAgene™ Dx product handbook. DNA Genotek. PD-HB-1. [PDF].

Warning

For In Vitro Diagnostic Use

DNAGENOTEK™

*Superior samples
Proven performance*

DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

 QbD RepS BV, Groenenborgerlaan 16, 2610 Wilrijk, Belgium

UK Responsible Person: Qarad UK Ltd., 8 Northumberland Ave, Westminster, London, WC2N 5BY United Kingdom

 Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Switzerland

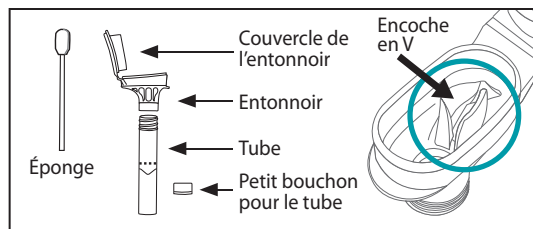
Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

OAgene and DNA Genotek are trademarks of DNA Genotek Inc., registered under various jurisdictions. Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legal-notices)

© 2025 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

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Odgovorna oseba v ZK: Qarad UK Ltd., 8 Northumberland Ave, Westminster, London, WC2N 5BY Združeno kraljestvo
CH REP Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Švica
© 2025 DNA Genotek Inc., podružnica OraSure Technologies, Inc., vse pravice pridržane.