
EN English – User instructions	2
FR Français – Mode d’emploi	3
ES Español – Instrucciones de uso	4
DE Deutsch – Gebrauchsanweisung	5
NB Norsk – Bruksanvisning	6
SV Svenska – Användarinstruktioner	7
DA Dansk – Brugsanvisning	8
NL Nederlands – Gebruiksaanwijzing	9
CS Český – Pokyny pro uživatele	10
IT Italiano – Istruzioni per l’utente	11
PL Polski – Instrukcja obsługi	12
PT Português – Instruções do utilizador	13
BZ Português (Brasil) – Instruções de uso	14
AR العربية – تعليمات المستخدم	15
ZH 中文 – 使用说明	16
KO 한글 – 사용 설명서	17
JA 日本語 – 取扱説明書	18
BG Български – Инструкции за потребителя	19
HR Hrvatski – Korisničke upute	20
ET Eesti keel – Kasutusjuhend	21
FI Suomi – Käyttöohjeet	22
EL Ελληνικά – Οδηγίες χρήστη	23
HU Magyar – Használati utasítás	24
IE Gaeilge – Treoracha úsáideora	25
LV Latviešu valoda – Lietotāja pamācība	26
LT Lietuvių k. – Naudotojo instrukcijos	27
MT Malti – Istruzzjonijiet għall-utent	28
RO Română – Instrucțiuni de utilizare	29
RU Русский – Инструкции для пользователя	30
SK Slovensky – Návod na použitie	31
SL Slovenščina – Navodila za uporabo	32
TH ภาษาไทย – คำแนะนำสำหรับผู้ใช้	33
CY Cymraeg – Cyfarwyddiadau i ddefnyddwyr	34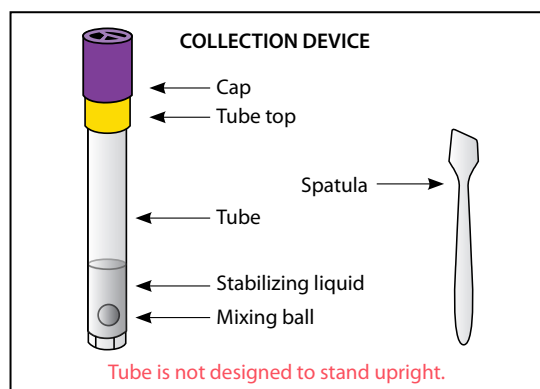

Intended use/purpose: Intended for the collection and stabilization of microbial DNA from human fecal samples.

Intended user: This product is intended for use by adults.

Contents: 1 kit containing 2 mL stabilizing liquid.

Summary and explanation of the kit:

OMNigene™ GUT provides the materials and instructions for collecting and stabilizing microbial DNA from a fecal sample.

Warnings and precautions:

- FOR EXTERNAL USE ONLY.
- Do NOT remove the yellow tube top from the tube.
- Do NOT spill the stabilizing liquid in the tube.
- Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.
- If collecting a liquid fecal sample, see separately provided user instructions.
- Small items may pose a choking hazard.
- Report any serious incident to DNA Genotek and the competent authority in your country

Storage: Room temperature storage pre- and post-collection 15 °C / 25 °C (59 °F / 77 °F).

Label legend:

	Flammable liquid
	Exclamation mark (Warning)
	Manufacturer
	Collect sample by (Use by)
	Catalog number
	Lot number
	CE marking
	UKCA marking
	Do not re-use
	Consult package insert
	In vitro diagnostic medical device
	Caution, consult instructions for use
	European Authorized Representative
	Swiss Authorized Representative
	Unique Device Identifier
	Quantity of units per package
	Storage instructions

15 °C / 25 °C
59 °F / 77 °F

USER INSTRUCTIONS

Read all instructions prior to collection.

Other languages are available at www.dnagenotek.com


Procedure:


- IMPORTANT PREPARATIONS:**
 - Empty your bladder before beginning the collection.
 - Collect fecal sample free of urine or toilet water.
 - Utilize any toilet accessory which may be supplied with the collection device.
 - Toilet paper or tissues may be required.

- While holding the yellow tube top, unscrew ONLY the purple cap and set aside for later use.

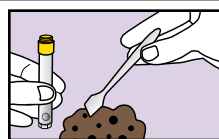
- Use the spatula to collect a small amount of fecal sample.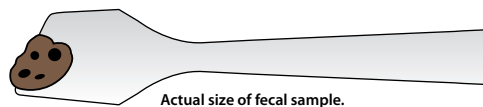


- Transfer the fecal sample into the top portion of the yellow tube top. Repeat until the sample fills the yellow tube top.

IMPORTANT: Do NOT push sample into the tube.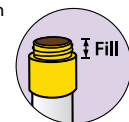


- Scrape horizontally across the tube top to level the sample and remove any excess.


Wipe exterior of tube and top with toilet paper or tissue as needed.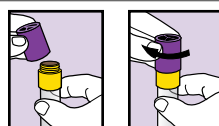

- Pick up the purple cap and screw onto the yellow tube top until tightly closed.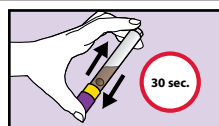

- For a **minimum of 30 seconds**, shake the sealed tube as hard and fast as possible in a back and forth motion.

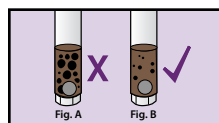- The fecal sample will be mixed with the stabilizing liquid in the tube; not all particles will dissolve.

IMPORTANT: Continue shaking if **large** particles remain as shown in Figure A.

- Place spatula in original packaging or wrap in toilet paper and discard in garbage.

IMPORTANT: Send the sample for processing following the delivery instructions supplied separately by the kit provider.

Ship in accordance to applicable regulations covering transport of biological specimens.

Warning

CE UKCA IVD

For In Vitro Diagnostic Use

DNAGENOTEK™

*Superior samples
Proven performance*

DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

UK Responsible Person: Qarad UK Ltd., 8 Northumberland Ave, Westminster, London, WC2N 5BY United Kingdom

EC REP QbD RepS BV, Groenenborgerlaan 16, 2610 Wilrijk, Belgium

CH REP Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Switzerland

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

OMNigene™ GUT (OM-200) is not available for sale in the United States.
OMNigene and DNA Genotek are trademarks of DNA Genotek Inc., registered under various jurisdictions.
Some DNA Genotek products may not be available in all geographic regions; contact your sales representative for details.
All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)

© 2025 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

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Person Cyfrifol y DU: Qarad UK Ltd., 8 Northumberland Ave, Westminster, Llundain, WC2N 5BY Y Deyrnas Unedig
CH REP Qarad Suisse S.A., World Trade Center, Avenue Gratta-Paille 2, 1018 Lausanne, Y Swistir
© 2025 DNA Genotek Inc., is-gwmni i OraSure Technologies, Inc., cedwir pob hawl.