EN ENGLISH
User instructions

Read all instructions prior to collection.

Collection precautions:

Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Donor should NOT eat, drink, smoke, or chew gum for 30 minutes before collecting oral sample.

Intended use: This product is designed for the collection and stabilization of DNA from human oral samples

Contents: 1 kit containing stabilizing liquid

Warnings and precautions:

Choking hazard. Caution should be used when inserting sponge into the mouth.

Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C | 25°C (59°F | 77°F)

Summary and explanation of the kit:

ORACollect-DNA is a self-collection kit that provides the materials and instructions for collecting human oral samples.

Instructions for sample collection:

1 Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.

2 Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.

3 Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.

4 Turn the cap upside down, insert the sponge into the tube and close cap tightly.

5 Invert the capped tube and shake vigorously 15 times.

Label legend:

REF Catalog number

MD In vitro diagnostic medical device*

CE CE marking

UKCA UKCA marking

MD Manufacturer

MD Medical device*

Ⓢ Do not reuse

Ⓢ Consult instructions for use

Ⓢ Use by

LOT Lot number

EU European Authorized Representative

CH Swiss Authorized Representative

UDI Unique Device Identifier

15°C | 25°C Storage instructions
59°F | 77°F

*Per applicable regulatory requirements, this device may be a medical device or in vitro diagnostic medical device.

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