
**Collection precautions:**

Ensure the sponge tip does **NOT** come into contact with any surface prior to collection.

Donor should **NOT** eat, drink, smoke or chew gum for 30 minutes before collecting oral sample.

**Intended use:** This product is designed for the collection and stabilization of DNA from human oral samples.

**Contents:** Contains 1 collection kit











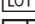
**Warnings and precautions:** Choking hazard. Caution should be used when inserting sponge into the mouth.

Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at [www.dnagenotek.com](http://www.dnagenotek.com)

**Storage:** 15°C-25°C

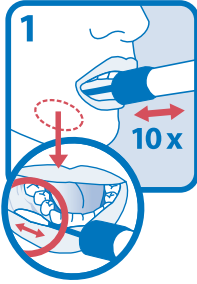
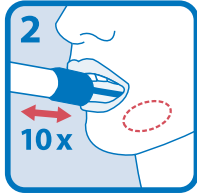


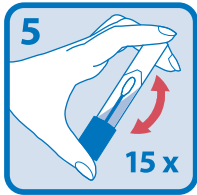
**Summary and explanation of the kit:** ORACollect-DNA is a self-collection kit that provides the materials and instructions for collecting human oral samples.

**Label legend:**

	Catalog number
	In vitro diagnostic medical device
	CE marking
	UKCA marking
	Manufacturer
	Medical device
	Do not re-use
	Consult instructions for use
	Use by
	Lot number
	Authorized Representative

## ENGLISH

**Instructions for sample collection:**

	<p>Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.</p>
	<p>Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.</p>
	<p>Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.</p>
	<p>Turn the cap upside down, insert the sponge into the tube and close cap tightly.</p>
	<p>Invert the capped tube and shake vigorously 15 times.</p>