


USER INSTRUCTIONS

Read all instructions prior to collection.

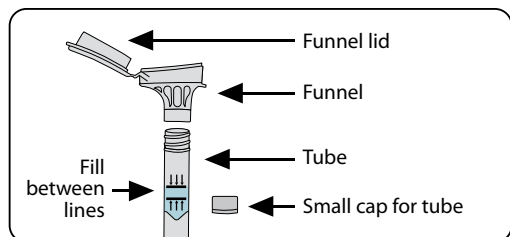

Intended use/purpose: For the collection of human DNA from saliva samples.

Intended user: This product is intended for use in adults.

Contents: 1 kit containing stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See MSDS at www.dnagenotek.com.

Small cap may pose a choking hazard.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C / 30°C (59°F / 86°F)

Summary and explanation of the kit:

Oragene•ONE is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.
















Collection precautions:

Do NOT eat, drink, smoke, or chew gum for 30 minutes before giving your saliva sample.

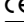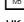Do NOT remove the plastic film from the funnel lid.


Do NOT spit above the top line on the tube (picture 1).

Label legend:

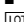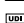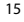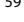
	Flammable liquid
	Catalog number
	CE marking
	In vitro diagnostic medical device
	UKCA marking
	Do not re-use
	Consult package insert
	Caution, consult instructions for use
	Manufacturer
	European Authorized Representative
	Swiss Authorized Representative
	Collect saliva by (Use by)
	Lot number
	Unique Device Identifier
	Storage instructions

Procedure:


Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 5.

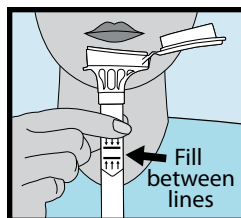


- 1 Spit a SMALL amount of saliva into the funnel. Gently tap the base of the tube on a flat surface to help the saliva move into the tube. Check that liquid saliva (not bubbles) level is between the lines as shown in picture #1.

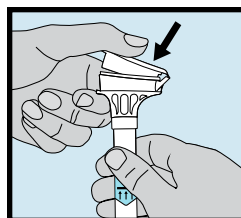


- 2 Hold the tube upright with one hand. Close the funnel lid with the other hand (as shown) by firmly pushing the lid until you hear a loud click. The liquid in the lid will be released into the tube to mix with the saliva. Make sure that the lid is closed tightly.

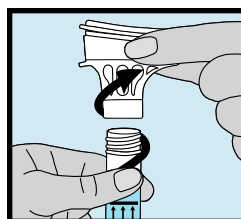

- 3 Hold the tube upright. Unscrew the funnel from the tube.

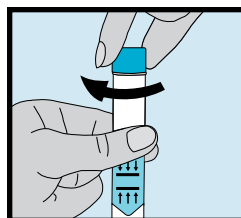

- 4 Use the small cap to close the tube tightly.

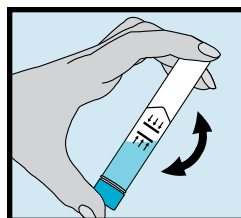

- 5 Shake the capped tube for 5 seconds. Discard or recycle the funnel.



For In Vitro Diagnostic Use

DNAGENOTEK™

*Superior samples
Proven performance*

Made in Canada
DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

UK Responsible Person: Emergo Consulting (UK) Limited c/o Cr360 - UL International, Compass House, Vision Park Histon, Cambridge, CB24 9BZ

 Novosanis NV Bijkhoevelaan 32c, 2110 Wijnegem, Belgium Email: EUAR@novosanis.com

 Arazy Group Swiss GmbH, Bruderholzallee 53, 4059 Basel, Schweiz Email: swiss.ar@arazygroup.com
Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

Oragene•DNA is not available for sale in the United States.

Oragene is a registered trademark and DNA Genotek is a trademark of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions, contact your sales representative for details. All DNA Genotek protocols, white papers and application notes are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)

© 2022 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

PD-PR-00260 Issue 6/2022-05