Read all instructions prior to collection.

Intended use: For the collection of human DNA from saliva samples.

Contents: Kit contains stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. Do not ingest. See MSDS at www.dnagenotek.com.

Storage: 15°C / 25°C

Summary and explanation of the kit:

Oragene-DNA is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Collection precautions:

Do NOT eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.

Do NOT remove the plastic film from the cap.

Procedure:

Most people take between 2 and 5 minutes to deliver a saliva sample following steps 1 to 4.

- 1 Start giving your saliva sample.
- 2 Spit until the amount of liquid saliva (not bubbles) reaches the level shown in picture #2.
- 3 Put the container on a flat surface. Screw the cap onto the container. Make sure that the cap is closed tightly, as shown in picture #3c. If you have difficulty closing the cap completely, turn cap slightly counter-clockwise then try again.
- 4 Shake gently for 10 seconds. Recycle outer packaging.

Label legend:

- Consult package insert
- Collect saliva by (Use by)
- In vitro diagnostic medical device
- Catalog number
- CE Marking
- Caution, consult instructions for use
- Storage instructions
- Authorized Representative
- Manufacturer
- Lot number

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