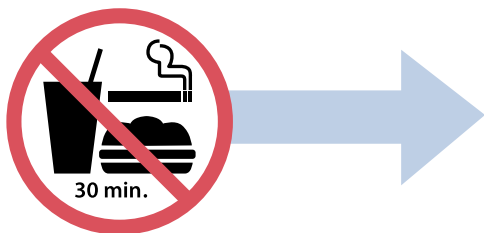
English – Instructions for sample collection	2
Français – Instructions de prélèvement de l'échantillon	3
Español – Instrucciones para la recogida de la muestra.....	4
Deutsch – Anweisungen zur Probenahme	5
Norsk – Instruksjoner for prøveinnsamling	6
Svenska – Anvisningar för provinsamling	7
Dansk – Instruktions til indsamling af prøve	8
Nederlands – Instructies voor monstername.....	9
Česky – Pokyny k odběru vzorku.....	10
Italiano – Istruzioni per la raccolta dei campioni	11
Polski – Instrukcje dotyczące pobierania próbek.....	12
Português – Instruções para a recolha de amostras	13
Português – Brasil – Instruções para coleta de amostras	14
العربية – إجراء جمع عينة للعباللعاب	15
中文 – 样本采集说明	17
한글 – 타액 샘플 채취용 절차	18
日本語 – サンプル採取に関する指示書	19
Български – Инструкции за вземане на проба	20
Hrvatski – Upute za prikupljanje uzoraka	21
Eesti keel – Juhised proovivõtmiseks	22
Suomi – Ohjeet näytteen ottamiseen	23
Ελληνικά – Οδηγίες συλλογής δείγματος	24
Magyar – Mintagyűjtési utasítások.....	25
Gaeilge – Treoracha maidir le bailiú an sampla.....	26
Latviešu valoda – Norādījumi parauga paņemšanai	27
Lietuvių k. – Mėginio paėmimo instrukcijos.....	28
Malti – Istruzzjonijiet għall-ġbir ta' kampjun	29
Română – Instrucțiuni pentru colectarea probelor	30
Русский – Инструкции для сбора образца	31
Slovensky – Pokyny na odber vzorky	32
Slovenščina – Navodila za jemanje vzorca:	33
עברית – הליך לאיסוף דגימת רוק	34

Collection precautions:

Ensure the sponge tip does **NOT** come into contact with any surface prior to collection.

Donor should **NOT** eat, drink, smoke or chew gum for 30 minutes before collecting oral sample.

Intended use: This product is designed for the collection and stabilization of DNA from human oral samples.

Contents: 1 kit containing 1 mL stabilizing liquid

Warnings and precautions: Choking hazard. Caution should be used when inserting sponge into the mouth.

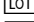
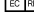
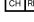
Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C.↕25°C (59°F.↕77°F)

Summary and explanation of the kit: ORACollect-DNA is a self-collection kit that provides the materials and instructions for collecting human oral samples.

Label legend:

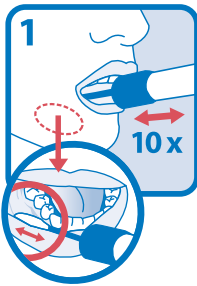
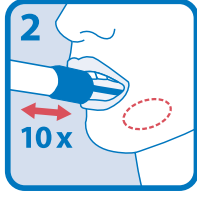
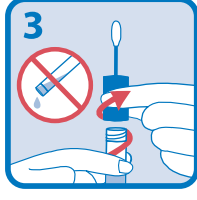
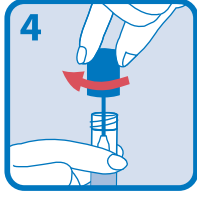
	Catalog number
	In vitro diagnostic medical device**
	CE marking
	UKCA marking
	Manufacturer
	Medical device*
	Do not reuse
	Consult instructions for use
	Use by
	Lot number
	European Authorized Representative
	Swiss Authorized Representative
	Unique Device Identifier

15°C.↕25°C Storage instructions
59°F.↕77°F

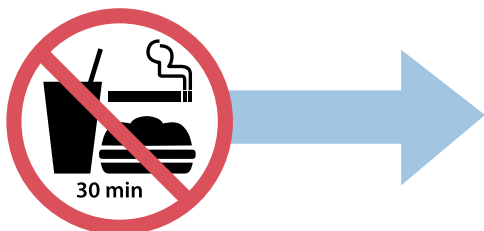
Per applicable regulatory requirements, the device contains a swab component, which is a *medical device, and a collection tube component, which is an **in vitro device.

ENGLISH

Instructions for sample collection:

	<p>Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.</p>
	<p>Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.</p>
	<p>Hold the tube upright to prevent the stabilizing liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.</p>
	<p>Turn the cap upside down, insert the sponge into the tube and close cap tightly.</p>
	<p>Invert the capped tube and shake vigorously 15 times.</p>

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© 2024 DNA Genotek Inc., uma subsidiária da OraSure Technologies, Inc., todos os direitos reservados.
DNA Genotek Inc.