

Resúmen i splikashon di e kaha/set:

OMNIGENE-GUT ta duna e materialnan i instruksjonnan pa kolektá i stabilisá DNA di mikrobio for di e muestra di materia fecaal.






Rekomendashon i prekoushonnan:

- SOLAMENTE PA USO EKSTERNO.
- NO kita e parti ariba (tòp) di e tubu hel for di e tubu.
- NO basha e líkido stabilisado den e tubu.
- Laba ku awa si e líkido a bai den wowo of riba kueru. NO guli e líkido.
- Si e muestra di materia fecaal ta líkido, lesa e otro instruksjon inkluí.
- Partinan chikitu por forma un rísiko pa chokamentu.

Warda e produkto na un temperatura di: 15°C pa 25°C

Post di akuerdo ku regulashonnan aplikabel pa kubri transportashon di espesienan biológiko. Chek MSDS na www.dnagenotek.com

Informashon riba e etikèt:

-  Kolektá muestra promé ku (Usa promé ku)
-  Number di katalóg
-  Fabrikadó
- 15°C / 25°C Instruksjonnan pa warda produkto
-  Koutela, konsultá instruksjonnan di uso
-  Number di porshon

INSTRUKSHONAN DI USO

Lesu tur instruksjon promé ku kolekshon di muestra di materia fecaal.

Prosedura:



- 1 PREPARASHONAN IMPORTANTE:**
- Hasi bo blas bashí promé ku bo kuminsá ku e proseso di kolekshon.
 - Kolektá e muestra di materia fecaal sin orina of awa di wc.
 - Papel di wc o klineks lo por ta nesesario.



- 2** Tene e parti ariba (tòp) di e tubu hel. Despues kita e skruf di e dòpi lila SO for di e kaha/set I pone esaki un banda pa hasi uso di esaki despues.



No lagé basha

IMPORTANTE:


NO kita e parti ariba (tòp) di e tubu hel. NO basha e líkido stabilisado den e tubu.



- 3** Usa e spatula pa kolektá un kantidat chikitu di e muestra di materia fecaal.



Grandura aktual/real di e muestra di af.

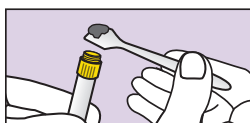


- 4** Pasa e muestra di e materia fecaal den e parti ariba (tòp) di e tubu hel.

IMPORTANTE: NO pusha e muestra den e tubu.



- 5** Skrap den forma stret (horizontal) riba e tòp di e tubu pa nivelá e muestra i eliminá/kita kualke ekseso.


Seka e parti pafo di e tubu i e tòp ku papel di wc of klineks, si ta nesesario.

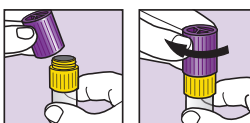

- 6** Kue e dòpi lila ku e parti sólido ku direkshon abou i skruf esaki tait sera riba e parti ariba (tòp) di e tubu hel.




- 7** Sakudí e tubu, seyé mas duru I lihé posibel den un moveshon di patras bai dilanti no menos ku 30 sekonde.

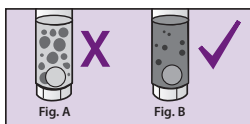

- 8** E muestra di materia fecaal lo wòrdu meskla ku e líkido stabilisadó den e tubu.

IMPORTANTE: Sigui sakudí si partinan grandi keda presente, manera indiká den Figura A.

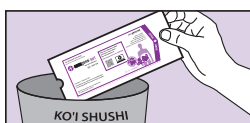

- 9** Pone e spatula den e pakete original o lora esaki den papel di wc i tira esaki den ko'i shushi.

IMPORTANTE: Post e muestra pa e wòrdu prosená, siguiendo e instruksjonnan ku e proveedor di e kaha/set a duna.