


OM-AC2 Spoon accessory

Used with  **OMNigene®-GUT**

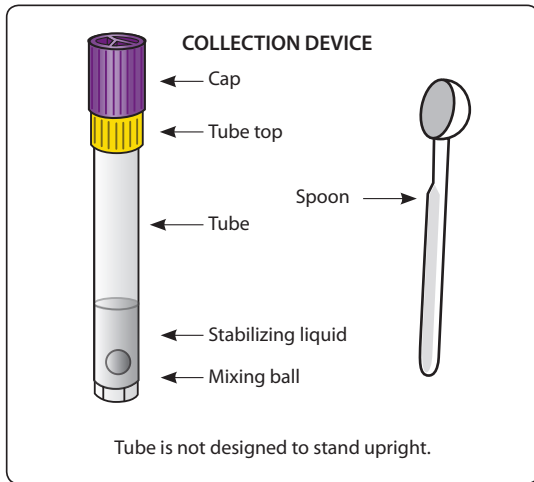

Preparation:

- Empty your bladder before beginning the collection.
- Collect fecal sample free of urine or toilet water.
- Toilet paper or tissues may be required.

Summary and explanation of the kit:

OMNigene®-GUT with spoon accessory provides the materials and instructions for collecting and stabilizing microbial DNA from liquid fecal sample (severe diarrhea).






Warnings and precautions:

- FOR EXTERNAL USE ONLY.
- Do NOT remove the yellow tube top from the tube.
- Do NOT spill the stabilizing liquid in the tube.
- Wash with water if liquid or sample comes in contact with eyes or skin. Do NOT ingest.
- Small items may pose a choking hazard.

Storage: 15°C to 25°C

Ship in accordance to applicable regulations covering transport of biological specimens. See MSDS at www.dnagenotek.com


Label legend:

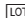-  Collect sample by (Use by)
-  Catalog number
-  Manufacturer
- 15°C / 25°C Storage instructions
-  Caution, consult instructions for use
-  Lot number

USER INSTRUCTIONS

Read all instructions prior to collection

Procedure:

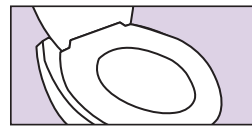

- 1 IMPORTANT PREPARATIONS:**
- Empty your bladder before beginning the collection.
 - Collect fecal sample free of urine or toilet water.
 - Toilet paper or tissues may be required.



- 2** While holding the yellow tube top, unscrew ONLY the purple cap from the kit and set aside for later use.




IMPORTANT:
Do NOT remove the yellow tube top.
Do NOT spill the stabilizing liquid in the tube.



- 3** Transfer **one spoonful** of liquid fecal sample into the yellow tube top.


IMPORTANT:
Fecal sample may not flow through the yellow tube top. Proceed to step #4.

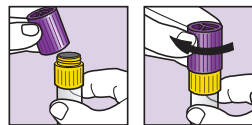

- 4** Pick up the purple cap with the solid end facing down and screw onto the yellow tube top until tightly closed.




- 5** Shake the sealed tube as hard and fast as possible in a back and forth motion for a minimum of 30 seconds.

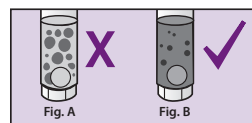

- 6** The fecal sample will be mixed with the stabilizing liquid in the tube; not all particles will dissolve.

IMPORTANT: Continue shaking if large particles remain as shown in Figure A.

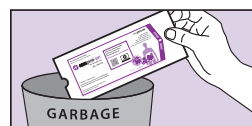

- 7** Place spoon in original packaging or wrap in toilet paper and discard in garbage.

IMPORTANT: Send the sample for processing following the delivery instructions supplied by the kit provider.

DNA genotek



Superior samples
Proven performance

Made in Canada
DNA Genotek Inc.
3000 - 500 Palladium Drive
Ottawa, ON, Canada K2V 1C2

Toll-free (North America): 1.866.813.6354
Tel.: +1.613.723.5757 • Fax: +1.613.723.5057
info@dnagenotek.com
www.dnagenotek.com

*OMNigene is a registered trademark of DNA Genotek Inc.

Some DNA Genotek products may not be available in all geographic regions, contact your sales representative for details. All DNA Genotek protocols, white papers and application notes, are available in the support section of our website at www.dnagenotek.com.

Patent (www.dnagenotek.com/legalnotices)
© 2017 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.
PD-PR-00763 Issue 1/2017-11