**DNAgenotek™**

REF OGD-575/OGD-675
DNA Genotek Inc.
 3000 - 500 Palladium Drive
 Ottawa, ON, Canada K2V 1C2


Tel.: +1.613.723.5757
 info@dnagenotek.com
 www.dnagenotek.com

Patent (www.dnagenotek.com/legalnotices)
 PD-PR-00941 Issue 4/2023-12

Warning

CE IVD MD UK CA

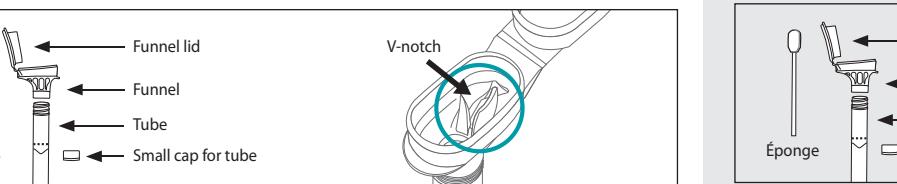ENGLISH**User instructions**

For In Vitro Diagnostic Use
 For Prescription and Direct-to-Consumer Use

Read all instructions prior to collection.

Intended use: For the assisted collection of human DNA from saliva samples.

Contents: 1 kit containing 0.75 mL stabilizing liquid

Warnings and precautions:

- Choking hazards:
 - Small cap in collection kit.
 - Plastic bag containing sponges.
 - Caution should be used when inserting sponge into donor's mouth.
- For supervised collections:
 - Do NOT leave donor unattended.
 - Do NOT allow donor to handle the sponge, small cap or packaging.
 - Wash with water if stabilizing liquid comes in contact with eyes or skin. Do NOT ingest. See SDS at www.dnagenotek.com.

Report any serious incident to DNA Genotek and the competent authority in your country.

Storage: 15°C - 30°C (59°F - 86°F)

Summary and explanation of the kit:
 Oragene®-Dx is an assisted collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Collection precautions:
Do NOT remove the plastic film from the funnel lid.

Check sponge for damage each time before inserting into donor's mouth. Use second sponge if first sponge shows any signs of wear or tear.

Do NOT substitute with other sponges or swabs.

See additional instructions for use PD-HB-00001 at www.dnagenotek.com

Label legend:

	Flammable liquid Catalog number
	In vitro diagnostic medical device**
	CE marking
	UKCA marking
	Manufacture
	Medical device*
	Do not reuse
	Consult instructions for use
	Collect saliva by (Use by)
	Lot number
	European Authorized Representative
	Swiss Authorized Representative
	Caution
	Unique Device Identifier
	Storage instructions
	15°C - 30°C
	59°F - 86°F
	Per applicable regulatory requirements, the device contains a swab component, which is a *medical device, and a collection tube component, which is an **in vitro device.

Superior samples • Proven performance


Novosanis NV, Bijkhoevelaan 32c,
 2110 Wijnegem, Belgium
 Email: EUAR@novosanis.com

UK Responsible Person: Emergo Consulting (UK) Limited c/o Cr360 – UL International, Compass House, Vision Park Histon, Cambridge, CB24 9BZ

Arazy Group Swiss GmbH
 Bruderholzallee 53, 4059 Basel, Schweiz
 Email: swiss.ar@arazygroupp.com

Australian Sponsor: Emergo Australia, Level 20, Tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000 Australia

© 2023 DNA Genotek Inc., a subsidiary of OraSure Technologies, Inc., all rights reserved.

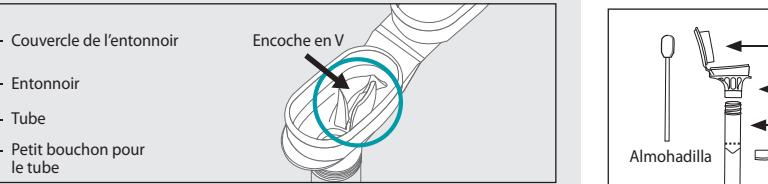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CH REF Arazy Group Swiss GmbH
© 2023 DNA Genotek Inc., uma subsidiária da OraSure Technologies, Inc.; todos os direitos reservados.